ORTHESE DYNAMIQUE DYNAMIC HAND-FINGER SPLINT

REF **W 332**

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Device Description

This device is designed to allow active finger flexion while keeping the wrist and fingers in passive extension. The device is intended only for the current indications and requires precise body measurements.

Device Content

Textile Components:

Polyamide – Polyester – PU Spong

Rigid Components:

Polypropylene – Aluminium – Steel

Device Application

Place the orthosis on the back of your hand and tighten the wrist straps one after the other, adjusting them to achieve a firm and comfortable fit. Insert your fingers into the sleeves and adjust for additional comfort.

Indications

- To correct muscle inactivity
- In ulnar nerve injuries
- Extensor tendon injuries
- Post-surgery rehabilitation

Contraindications

- Do not use the product when the diagnosis is uncertain.
- Do not allow the product to come into direct contact with injured skin.
- Do not use if allergic to any component.
- Do not use in cases of venous or lymphatic disorders in the neck region.

Important Information & Warnings

Inspect the device for damage before each use. Do not use if damaged.

Use under the supervision of a healthcare professional. Do not use the device while driving.

This device is for single-patient use only and should not be used by another person.

In case of discomfort, pain, abnormal swelling or discoloration of the skin, remove the device and consult a healthcare professional. The duration and frequency of daily use should be determined by a healthcare professional.

Avoid contact with oil-based substances such as ointments, creams, and lotions.

Device Maintenance & Storage

Hand wash in warm water with a mild detergent. Keep the device clean and store it in its original box when not in use.