ORTHESE DE REPOS STATIQUE STATIC WRIST BRACE SPLINT

REF **W 335**

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Device Description

This device is designed to keep the wrist and fingers in their resting position and to immobilize them in a neutral position. The device is intended only for the current indications and requires precise body measurements.

Device Content

Textile Components:

Polyamide – Polyester – PU Spong

Rigid Components:

Polypropylen

Device Application

Place your palm, thumb, and wrist inside the orthosis. Then tighten the wrist straps one after the other and adjust them to achieve a firm and comfortable fit. The thumb splint must be positioned without any painful pressure points. Next, adjust the thumb strap to obtain a firm and comfortable tightening.

Indications

- Carpal tunnel syndrome
- In cases of wrist and thumb calcification
- Tendon and soft tissue injuries
- Post-surgery rehabilitation

Contraindications

- Do not use the product when the diagnosis is uncertain.
- Do not allow the product to come into direct contact with injured skin.
- Do not use if allergic to any component.
- Do not use in cases of venous or lymphatic disorders in the neck region.

Important Information & Warnings

Inspect the device for damage before each use. Do not use if damaged.

Use under the supervision of a healthcare professional. Do not use the device while driving.

This device is for single-patient use only and should not be used by another person.

In case of discomfort, pain, abnormal swelling or discoloration of the skin, remove the device and consult a healthcare professional. The duration and frequency of daily use should be determined by a healthcare professional.

Avoid contact with oil-based substances such as ointments, creams, and lotions.

Device Maintenance & Storage

Hand wash in warm water with a mild detergent. Keep the device clean and store it in its original box when not in use.