ATTELLE DE DOIGT FINGER SPLINT

REF **W 339**

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Device Description

This device is a type of orthosis designed to immobilize the fingers. It should be used only for the intended indications and with precise body measurements adapted to the user.

Device Content

Textile Components:

Polyamide – Polyester

Rigid Components:

Aluminium

Device Application

After opening the device's Velcro straps, place your fingers inside and close the Velcro straps.

Indications

- Fractures of the fingers
- Tendon and nerve injuries
- Finger injuries and sprains

Contraindications

- Do not use the product when the diagnosis is uncertain.
- Do not allow the product to come into direct contact with injure skin.
- Do not use if allergic to any component.
- Do not use in cases of venous or lymphatic disorders in the neck region.

Important Information & Warnings

Inspect the device for damage before each use. Do not use if damaged.

Use under the supervision of a healthcare professional. Do not use the device while driving.

This device is for single-patient use only and should not be used by another person.

In case of discomfort, pain, abnormal swelling or discoloration of the skin, remove the device and consult a healthcare professional. The duration and frequency of daily use should be determined by a healthcare professional.

Avoid contact with oil-based substances such as ointments, creams, and lotions.

Device Maintenance & Storage

Hand wash in warm water with a mild detergent. Keep the device clean and store it in its original box when not in use.