## CORSET DORSOLOMBAIRE DORSOLUMBAR CORSET WITH STRAP

REF **W 406**

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### Device Description

Used for fractures, hernias, spinal slips, situations where the body needs to stay upright, and for posture correction. The device is intended only for the indications listed below and the size measurement.

### Device Content

#### Textile Components:

Polyamide – Polyester – Elastane

#### Rigid Components:

Polypropylene - Steel

### Device Application

Place the belt around the waist and fasten it at the abdomen using the hook-and-loop straps. The additional straps that go over the shoulders must be crossed at the back, then pulled forward. They are secured with Velcro to adjust the brace comfortably to the body.

### Indications

- In all vertebral hernias
- Sudden and long-term low back pain
- In osteoporosis
- In rehabilitation after surgical intervention

### Contraindications

- Do not use the product when the diagnosis is uncertain.
- Do not allow the product to come into direct contact with injure skin.
- Do not use if allergic to any component.
- Do not use in cases of venous or lymphatic disorders in the neck region.

### Important Information & Warnings

Inspect the device for damage before each use. Do not use if damaged.

Use under the supervision of a healthcare professional. Do not use the device while driving.

This device is for single-patient use only and should not be used by another person.

In case of discomfort, pain, abnormal swelling or discoloration of the skin, remove the device and consult a healthcare professional. The duration and frequency of daily use should be determined by a healthcare professional.

Avoid contact with oil-based substances such as ointments, creams, and lotions.

### Device Maintenance & Storage

Hand wash in warm water with a mild detergent. Keep the device clean and store it in its original box when not in use.