## CEINTURE ABDOMINALE EN NEOPRENE (24 cm) NEOPRENE ABDOMINAL CORSET (24 cm)

REF **W 413**

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### Device Description

This device provides support to the abdominal area. It prevents the sagging of the abdominal region after pregnancy. The device is intended only for the indications listed below and for the size measurement.

### Device Content

#### Textile Components:

Polyamide – Polyester – Neoprene – PU foam

#### Rigid Components:

### Device Application

Place the belt around the waist and close it at the abdomen using the hook-and-loop fasteners.

### Indications

- Postpartum recovery
- Support for the abdominal wall
- Rib fracture
- Rehabilitation after surgery
- Minor upper or lower back pain

### Contraindications

- Do not use the product when the diagnosis is uncertain.
- Do not allow the product to come into direct contact with injured skin.
- Do not use if allergic to any component.
- Do not use in cases of venous or lymphatic disorders in the neck region.

### Important Information & Warnings

Inspect the device for damage before each use. Do not use if damaged.

Use under the supervision of a healthcare professional. Do not use the device while driving.

This device is for single-patient use only and should not be used by another person.

In case of discomfort, pain, abnormal swelling or discoloration of the skin, remove the device and consult a healthcare professional. The duration and frequency of daily use should be determined by a healthcare professional.

Avoid contact with oil-based substances such as ointments, creams, and lotions.

### Device Maintenance & Storage

Hand wash in warm water with a mild detergent. Keep the device clean and store it in its original box when not in use.