## CEINTURE DE MATERNITE PREGNANCY BELT

REF **W 428**

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### Device Description

This device is designed to support women throughout their pregnancy by relieving lower back and pelvic pain. It is intended for use only according to the indications listed below and based on size measurements.

### Device Content

#### Textile Components:

Polyamide – Polyester – Elastane - PU foam

#### Rigid Components:

EVA- Polypropylene

### Device Application

Wrap the belt around the waist and fasten it at the abdomen using the self-gripping straps. Adjust the corset by tightening the side belts and securing them to the front of the belt. To use the additional belt included in the package, attach one side using the Velcro to the corset, then secure the other side with Velcro to the corset so that it passes over the abdomen.

### Indications

- Lower back pain related to pregnancy
- Pelvic pain related to pregnancy
- Sudden and persistent lower back pain

### Contraindications

- Do not use the product when the diagnosis is uncertain.
- Do not allow the product to come into direct contact with injured skin.
- Do not use if allergic to any component.
- Do not use in cases of venous or lymphatic disorders in the neck region.

### Important Information & Warnings

Inspect the device for damage before each use. Do not use if damaged.

Use under the supervision of a healthcare professional. Do not use the device while driving.

This device is for single-patient use only and should not be used by another person.

In case of discomfort, pain, abnormal swelling or discoloration of the skin, remove the device and consult a healthcare professional. The duration and frequency of daily use should be determined by a healthcare professional.

Avoid contact with oil-based substances such as ointments, creams, and lotions.

### Device Maintenance & Storage

Hand wash in warm water with a mild detergent. Keep the device clean and store it in its original box when not in use.