## GENOUILLERE ELASTIQUE SIMPLE KNEE SUPPORT

REF **W 502**

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### Device Description

This device is designed to support the knee joint and relieve pain associated with it. The device is intended only for the indications listed below and for size measurement.

### Device Content

**Textile Components:**

Polyamide – Polyester – PolyUrethane Foam

**Rigid Components:**

### Device Application

Bring your knee to a straight position, center the device on your knee, and fasten the Velcro straps.

### Indications

- Increased intra articular fluid
- In rehabilitation after surgical intervention
- Protection during and after sports activities.

### Contraindications

- Do not use the product when the diagnosis is uncertain.
- Do not allow the product to come into direct contact with injure skin.
- Do not use if allergic to any component.
- Do not use in cases of venous or lymphatic disorders in the neck region.

### Important Information & Warnings

Inspect the device for damage before each use. Do not use if damaged.

Use under the supervision of a healthcare professional. Do not use the device while driving.

This device is for single-patient use only and should not be used by another person.

In case of discomfort, pain, abnormal swelling or discoloration of the skin, remove the device and consult a healthcare professional. The duration and frequency of daily use should be determined by a healthcare professional.

Avoid contact with oil-based substances such as ointments, creams, and lotions.

### Device Maintenance & Storage

Hand wash in warm water with a mild detergent. Keep the device clean and store it in its original box when not in use.