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KNEE IMMOBILIZER

REF **W 519**

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Device Description

This device helps immobilize the knee and restrict the movement of the knee joint. The device is intended only for the indications listed below and for size measurement.

Device Content

Textile Components:

Polyamide – Polyester – PU Sponge

Rigid Components:

Polypropylene – Aluminum

Device Application

Fully open the front part of the brace. After placing your leg on the brace and making sure the kneecap is centered, tighten the straps to provide firm and comfortable support.

Indications

- Postoperaaative for meniscus injuries
- In lateral ligament damage
- Postoperative protection aand immobilizaation
- In rehabilitation after plaster cast

Contraindications

- Do not use the product when the diagnosis is uncertain.
- Do not allow the product to come into direct contact with injure skin.
- Do not use if allergic to any component.
- Do not use in cases of venous or lymphatic disorders in the neck region.

Important Information & Warnings

Inspect the device for damage before each use. Do not use if damaged.

Use under the supervision of a healthcare professional. Do not use the device while driving.

This device is for single-patient use only and should not be used by another person.

In case of discomfort, pain, abnormal swelling or discoloration of the skin, remove the device and consult a healthcare professional. The duration and frequency of daily use should be determined by a healthcare professional.

Avoid contact with oil-based substances such as ointments, creams, and lotions.

Device Maintenance & Storage

Hand wash in warm water with a mild detergent. Keep the device clean and store it in its original box when not in use.