BANDAGE DE TENDON ROTULIEN PATELLAR TENDON BAND

REF **W 520**

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Device Description

This device is designed to support the patellar tendon area located below the kneecap and to relieve knee-related pain. The device is intended only for the indications listed below and for size measurement.

Device Content

Textile Components:

Polyamide – Polyester – PU Sponge

Rigid Components:

Polypropylene – Silicone

Device Application

Place the device with its silicone support on the lower part of the kneecap, pass the strap through the buckle, and tighten.

Indications

- Patellar tendonitis
- Chondromalacia patella
- Femoral pain in the patella
- Os-good Schlatter disease

Contraindications

- Do not use the product when the diagnosis is uncertain.
- Do not allow the product to come into direct contact with injured skin.
- Do not use if allergic to any component.
- Do not use in cases of venous or lymphatic disorders in the neck region.

Important Information & Warnings

Inspect the device for damage before each use. Do not use if damaged.

Use under the supervision of a healthcare professional. Do not use the device while driving.

This device is for single-patient use only and should not be used by another person.

In case of discomfort, pain, abnormal swelling or discoloration of the skin, remove the device and consult a healthcare professional. The duration and frequency of daily use should be determined by a healthcare professional.

Avoid contact with oil-based substances such as ointments, creams, and lotions.

Device Maintenance & Storage

Hand wash in warm water with a mild detergent. Keep the device clean and store it in its original box when not in use.