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Device Description

This device helps stabilize the joints at the ankle. It is designed for the indications listed below and requires proper size measurement.

Device Content

Textile Components:

Polyamide – Polyester – Polyurethane foam

Rigid Components:

Polypropylene

Device Application

The device should be fitted while in a seated position. Loosen the straps and place the ankle between the two side shells. If necessary, adjust the width of the heel strap using the hook-and-loop fasteners located at the bottom of the side shells. Tighten the lower and upper straps and secure them properly with the hook-and-loop fasteners. Carefully put the foot with the device into the shoe.

Indications

- Chronic ankle instability
- Moderate to severe ankle sprain
- In ligament injuries
- Rehabilitation after surgery

Contraindications

- Do not use the product when the diagnosis is uncertain.
- Do not allow the product to come into direct contact with injured skin.
- Do not use if allergic to any component.
- Do not use in cases of venous or lymphatic disorders in the neck region.

Important Information & Warnings

Inspect the device for damage before each use. Do not use if damaged.

Use under the supervision of a healthcare professional. Do not use the device while driving.

This device is for single-patient use only and should not be used by another person.

In case of discomfort, pain, abnormal swelling or discoloration of the skin, remove the device and consult a healthcare professional. The duration and frequency of daily use should be determined by a healthcare professional.

Avoid contact with oil-based substances such as ointments, creams, and lotions.

Device Maintenance & Storage

Hand wash in warm water with a mild detergent. Keep the device clean and store it in its original box when not in use.